

WhisperReader user guide

Using the WhisperReader program, you can browse a medication reminder's alarms and historical information. The program is designed for browsing. Programming devices using WhisperReader is not possible.

Installation

After downloading the WhisperReader program, run the installation file (.exe) and follow the instructions. The WhisperReader program must be installed before connecting the Whisper medication reminder to the computer.

Use

1. Run WhisperReader
2. Connect the Whisper device. The alarms and the historical data of the device are visible in the calendar. You can browse the information but changes cannot be made. The time and date of the final alarm programmed into the device is shown on the right side of the screen.
3. Remove the device. The information is erased from the calendar but is saved in the device.
4. Connect another medication reminder or close the program.

